# Nutrio BP Electric

Each press of the increase/decrease button is indicated on the display. The default operating time is 30 minutes, after which the suction automatically switches off.

A single charge of the battery (2 hours) will enable 90 minutes of suction.

### Instructions for Use

1. Clean and sterilize all components of the breast pump before each use.
2. Make yourself comfortable and relax.
3. Place the suction cup on your breast and adjust the suction pressure to a comfortable level.
4. Ensure that the breast fits comfortably within the suction pad circumference.
5. After use, clean well and store away for future usage.

\* In case of difficulties, ensure that all components are correctly fitted.

### Maintenance

1. Disassemble all components when washing.
2. It is recommended to disinfect all parts of the device.
3. Carefully remove the valve from the silicone cap to avoid any damage. If the valve is damaged, the device will not function properly.
4. Use a soft fibre cloth to prevent scratches whilst cleaning any surfaces.
5. To sterilize we recommend the TrueLife Nutrio sterilizer.
6. Do not place in a dishwasher.
7. Do not place the motor unit in water. Wipe the display with a soft cloth.

### Information for the User

1. Always consult your doctor about using the breast pump.
2. Be especially careful when dismantling and storing the breast pump. Loss of individual parts prevents the device from functioning.
3. Keep the device away from children and minors.
4. Do not express milk if you feel pain, in order to prevent injury to the breasts.
5. Do not continue with use for more than 5 minutes if no milk is expressed.
6. If the pressure level is unpleasant, remove the breast pump from the breast.
7. Before use, check that the device fits well to your breast.
8. Wash and disinfect the device before use, clean and dry again after use.
9. Do not use abrasive or aggressive cleaning agents for cleaning.
10. The product is intended to be used by only one user.

### Storage and Feeding

1. You can store breast milk in the refrigerator for up to 48 hours, it is recommended to cool the milk immediately after expressing.
2. Breast milk can also be frozen and stored for up to 3 months.
3. Breast milk should only be stored in sterilized bottles or sterilized storage containers.
4. Label the containers in the refrigerator with the date of expiration.
5. Do not refreeze defrosted milk.
6. Do not mix fresh and thawed milk.
7. Heat milk in warm water or in a bottle heater.
8. Always check the milk's temperature before feeding.
9. Do not under any circumstances heat the milk in a microwave oven.
10. After feeding, any leftover milk should be poured away.

### Package Contents

- Control unit/Motor unit
- USB cable/Adapter
- Tube
- Breast pump
- Protective cap
- Silicone breast shield
- Teat
- Protective bottle cap
- Instruction Manual

### Manufacturer's Responsibility

TrueLife products are covered by a manufacturing warranty. It is the responsibility of Elem6 s.r.o. to repair or replace any defective parts or products for a comparable part or product at its sole discretion in the case of any warranty defects. Any other alteration or unauthorised manipulation of the product by outside parties exceeds the terms of this warranty. This also takes effect if the product is damaged as a result of defects caused by negligence or improper use. The warranty does not cover damage resulting from standard usage, breakage or from everyday wear and tear. Elem6 s.r.o. accepts no liability for any accidents, injuries, death, loss or other requirements related to or arising from the use of this product. Elem6 s.r.o. is not under any circumstances liable for incidental or consequential damages related to or resulting from the use of this product or any of its components. Full replacement or replacement of parts and/or products may be subject to shipping, handling or other fees.

Subject to printing errors and changes to information.

### SAFETY DURING USE:

1. Carefully read the User Manual before use.
2. For effective use of this product, read the most up-to-date version of the User Manual at: [www.truelife.eu](http://www.truelife.eu)
3. Subject to printing errors and changes to information.

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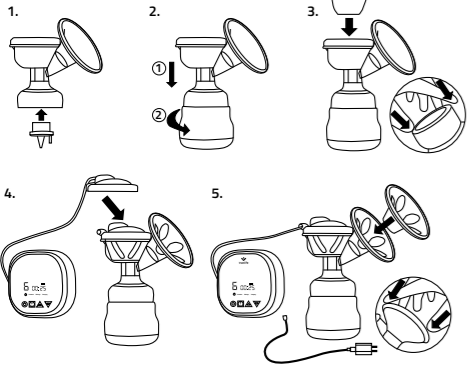
### Installation

\*Ensure that all components are cleaned and sterilized before use.

1. Fix the valve as shown. Incorrect fitting will cause malfunction as the valve prevents back-flow.
2. Twist the bottle onto the breast pump.
3. Attach the silicone diaphragm. Press the lower edges down into the aspirator so that it is firmly seated.
4. Attach the motor unit's tube to the breast shield cap. Press the cap down to close.
5. Attach the silicone breast shield to the suction cup. Make sure it is secure. Connect the adapter to the mains and choose the suction mode

### Suction

1. Press ON/OFF button (basic stimulation mode).
2. Press "Change mode" (stimulation/massage/suction).
3. Select stimulation intensity (9 levels).
4. Select suction intensity (9 levels).
5. Select massage intensity (9 levels).

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